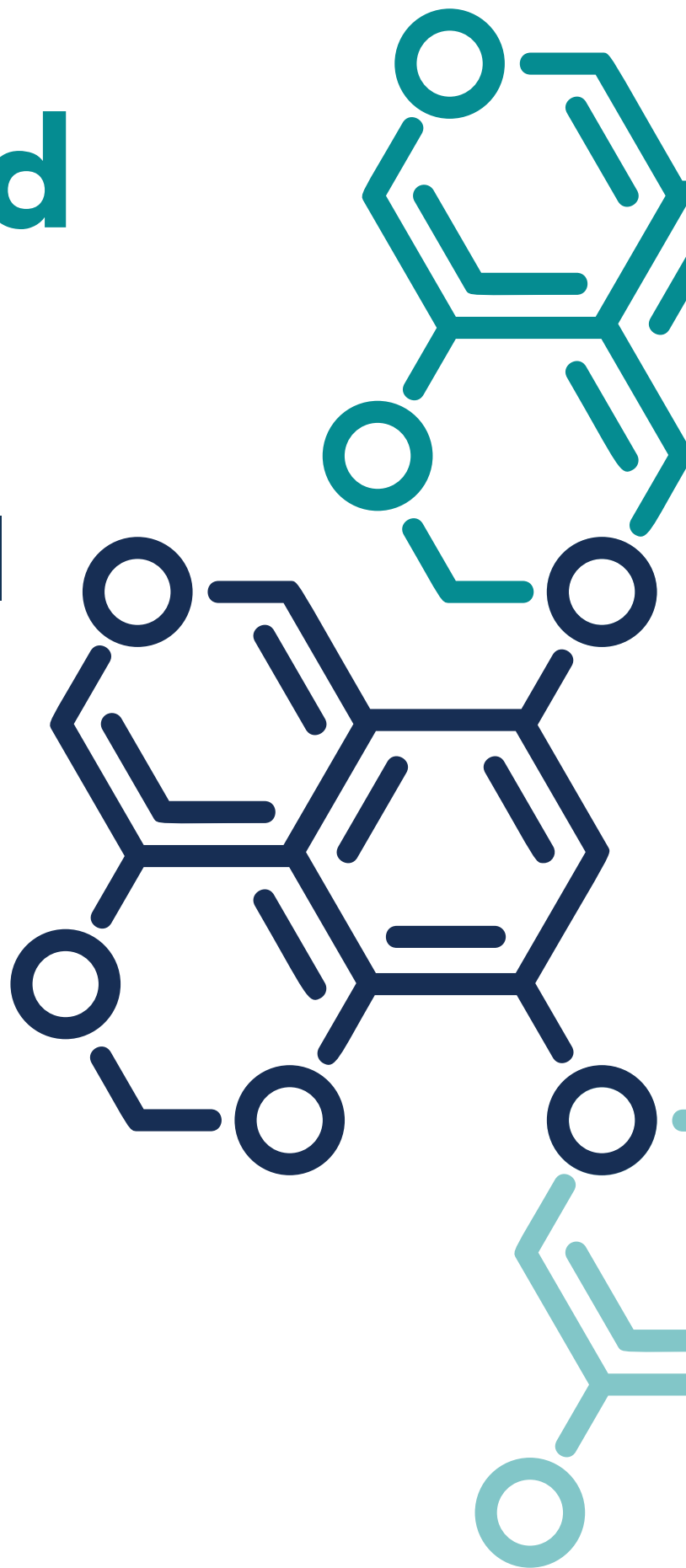
MTF

Metatarsal Spacer

Product description
& surgical technique

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Indications

The indication for use of Ø 16 mm MTF metatarsal spacer is surgery of the first metatarsophalangeal joint. More precisely, the device is indicated in case of:

- hallux rigidus;
- hallux valgus-rigidus;
- osteochondral lesion;
- metatarsophalangeal arthrosis;
- fractures outcome;
- rheumatic arthropaties.

The indication for use of Ø 13 mm MTF metatarsal spacer is surgery of the metatarsophalangeal joint of small toes. More precisely, the device is indicated in case of:

- osteochondral lesion;
- metatarsophalangeal arthrosis;
- fractures outcome;
- rheumatic arthropaties.

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Materials

PLLA in conformity with ASTM F1925 standard.

Materiali

PLLA conforme alla norma ASTM F1925.

Technical features

- Resorbable material, it avoids the implant removal surgery;
- Round shape of head to fit and restore the morphology of hallux or small toes metatarsal after resection of metatarsal head;
- Four reinforcement ribs in the undersurface of head positioned in cross configuration to enhance mechanical properties of the device;
- Shaft progressively tapers towards the bottom (conical shape), thus easing the insertion into the bone and the stability of the device by friction after insertion;
- Cannulated device to allow the insertion of a Kirschner wire aimed to obtain and keep an adequate metatarsophalangeal joint alignment;
- Radiolucent;
- Biocompatible.

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Surgical technique

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1st METATARSOPHALANGEAL JOINT

Medial or dorsal incision, 2 cm distally and 2 cm proximally to the articular line of metatarsophalangeal joint.

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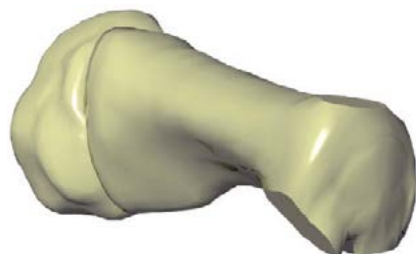
Removal of dorsal, medial and lateral osteophytes of metatarsal head.

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Regularization of any potential bone impingement between medial sesamoid and base of phalanx. Plantar release with detachment of glenoid labrum of the base of phalanx.

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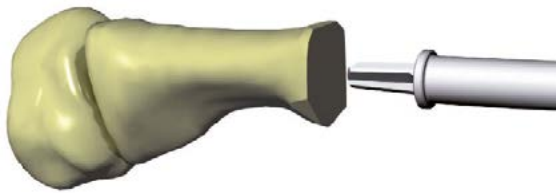
Resection of 3 mm of bone using oscillating saw with 5° oblique direction from top to bottom and proximal-distal.

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Surgical technique

Use the perforator MTF 060560 to create a hole in the center of metatarsal head, slightly undersized compared to the size of the spacer shaft.

Insertion of the metatarsal spacer by manual pressure or, in case of greater density bone, using the specific impactor MTF 071316.

Check joint congruence in flexion-extension.

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Surgical technique

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As an aid to keep a correct alignment between base and head, it is possible to insert a Ø 1.2 mm Kirschner wire in the center of the base of phalanx in proximal-distal direction, with wire that protrudes from the fingertip.

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The wire is inserted, in distal-proximal direction, on the spacer hole and along its shaft.

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The wire will be kept in position for two weeks until the removal of sutures and after that mobilization can start.

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Follow the same steps for the metatarsophalangeal joint of small toes surgeries.

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Bibliography

What's new in surgical options for hallux rigidus?

Giannini S, Ceccarelli F, Faldini C, Bevoni R, Grandi G, Vannini F.
J Bone Joint Surg Am. 2004;86-A Suppl 2:72-83.

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Magnan B, Samaila E, Viola G, Bartolozzi P.

First metatarsophalangeal joint replacement with total arthroplasty in the surgical treatment of the hallux rigidus.

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G.I.O.T. 2004;30:182-191.

Info

LSM-Med Srl
reserves the right
to make changes.

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OVERMED Srl