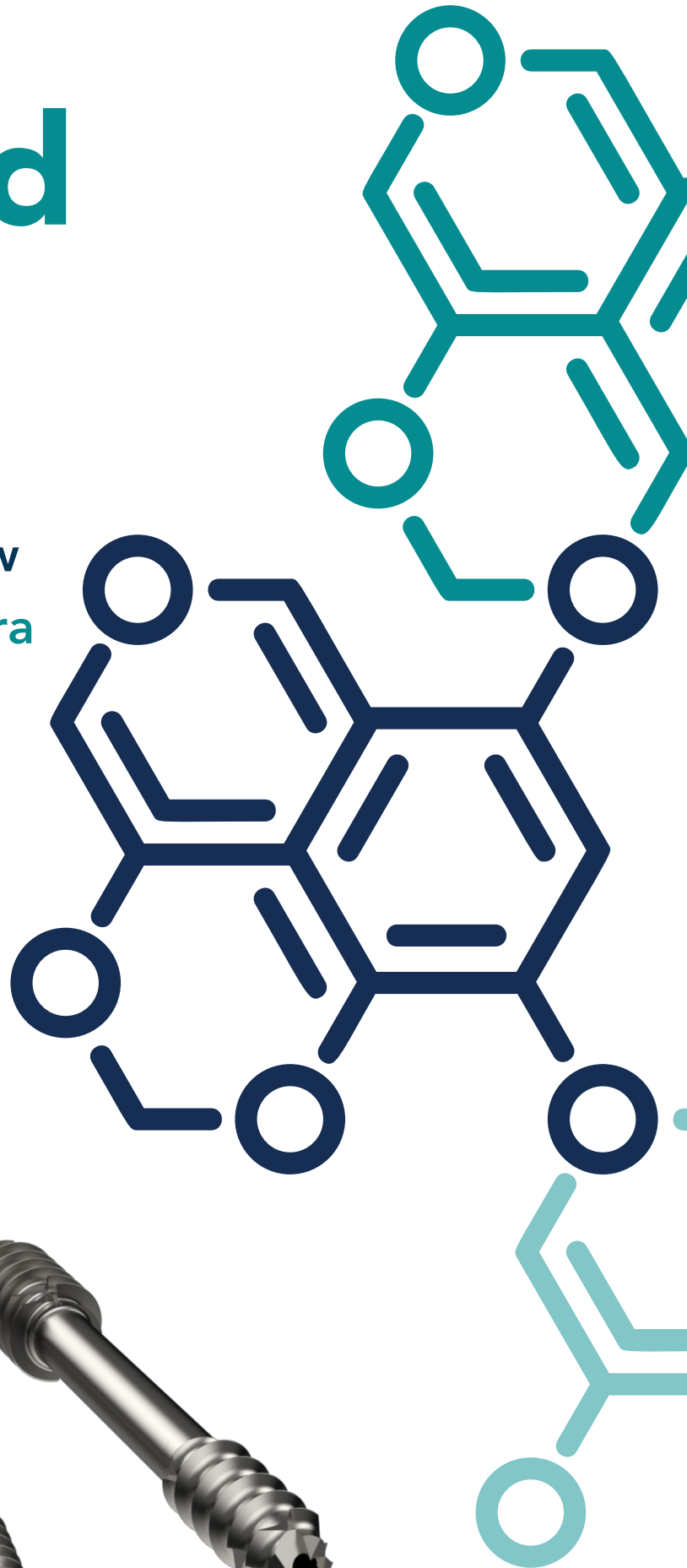
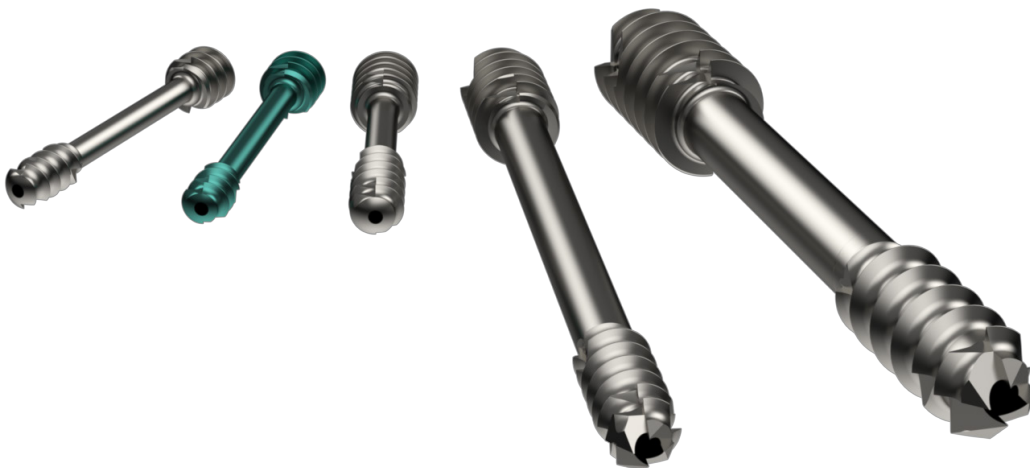
# DTS

Double threaded screw  
Vite a doppia filettatura

Product description  
& surgical technique

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## Indications

### DTS double threaded screws

Titanium cannulated screws with double differentiated pitch thread are suitable in the case of fractures where the lack of tissue makes the use of the traditional cancellous screw inappropriate or in those cases where a perfect fragment alignment is absolutely essential. The headless screw must be completely embedded in the bone thus avoiding any protrusion which could cause irritation tissue. Its special thread assures fracture compression and stability.

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## Materials

Ti6Al4V titanium alloy in conformity to the ISO 5832-3

## Materiali

Lega di titanio Ti6Al4V a norma ISO 5832-3

## Technical features

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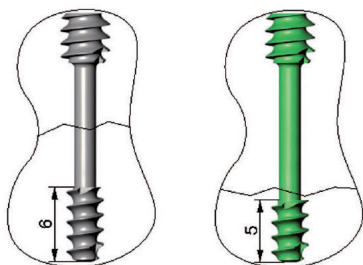
### DTS double threaded screws

- Self-tapping both in insertion and extraction
- Special differential thread for a better grip
- Less invasivity
- Wide range of diameters
- Special instrument set for scaphoid
- Scaphoid screw with standard and short thread.

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#### Distal pole - Polo distale

#### Proximal pole - Polo prossimale

In case of very small proximal pole, the head tip of the screw might not pass completely the fracture and lose its compressive effect. The special screws with short thread allow the solution of these problems.

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## Clinical indications

### Ø 3 - 3.5 mm DTS screws

- Scaphoid fractures
- Fixation of intra-articular and periarticular fragments
- Radial head fractures
- Carpal fusions
- Fractures of the first metacarpus

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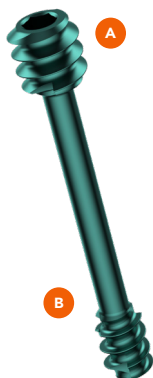
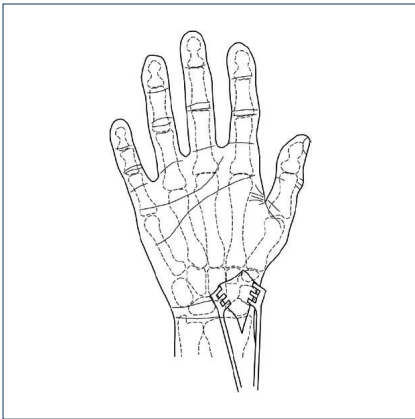
Ø Hegaxon / Esagono

## Surgical technique

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
### Surgical technique for ø 3 mm DTS screw

Anterior approach to scaphoid with exposure of the distal surface of the tubercle (transverse incision of the scaphotrapezial capsule).

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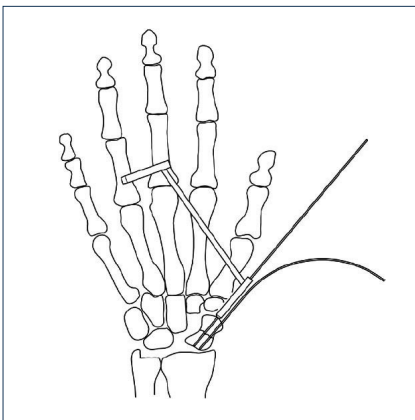
Eccentric insertion of a ø 0.8 mm kirschner wire along the radial margin of the scaphoid; this makes possible the simultaneously temporary fracture stabilization and the anterior scaphoid tilting movement to expose the ideal insertion point of the guide wire.

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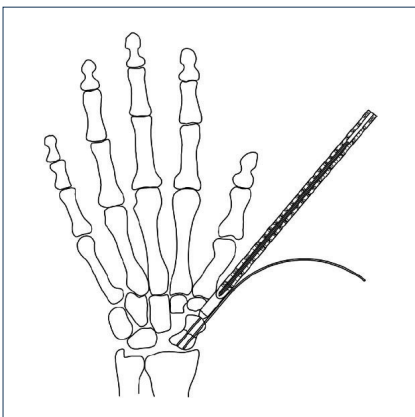
Determine the guide wire depth and therefore the screw size by using the proper measuring device.

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
## Surgical technique

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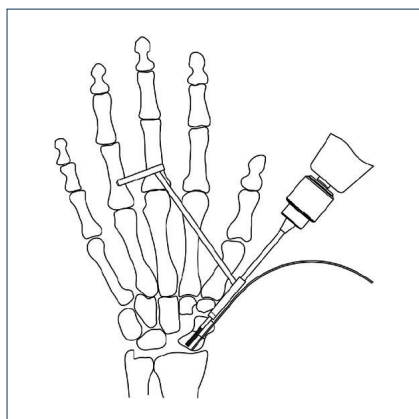Apply a traction on the kirschner wire and introduce the proper drill bit along the guide wire; the use of the guide for the drill bit is also possible.

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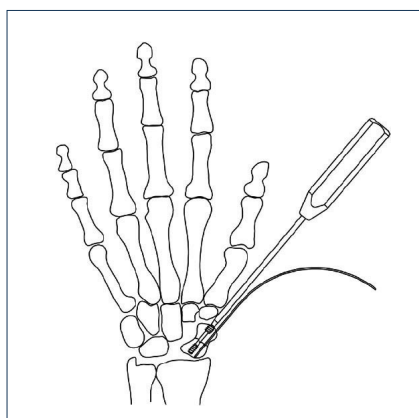If the guide wire is removed while extracting the drill bit, the screw can be inserted anyway or the guide wire can be reintroduced manually in the reamed canal.

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**Clinical indications****ø 4.5 - 6.5 mm DTS screws**

- Osteochondral and intra-articular fractures
- Fractures of the malleolar region
- Fractures of the olecranon
- Distal femoral fractures
- Pilon fractures
- Intercondylar fractures of the distal humerus
- Distal tibial fractures
- Arthrodesis of small joint
- Intercondylar femur fractures

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
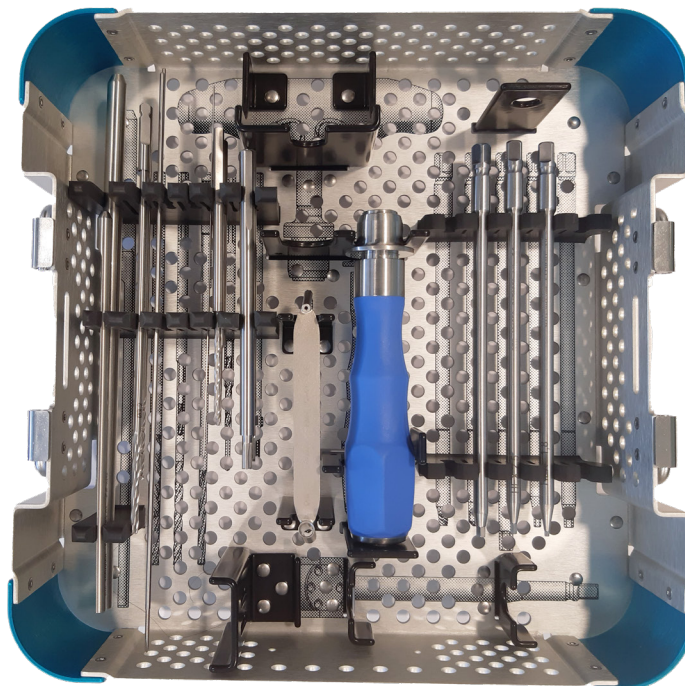
Ø Hegaxon / **Esagono**

**Surgical technique**

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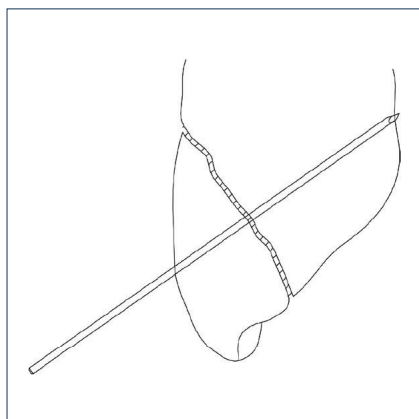
**Surgical technique for ø 4.5 - 6.5 mm DTS screw**

After the initial fragment reduction, insert the guide wire.

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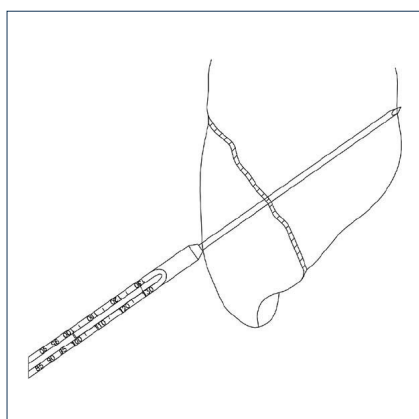
Determine the guide wire pdepth and therefore the screw size by using the proper measuring device.

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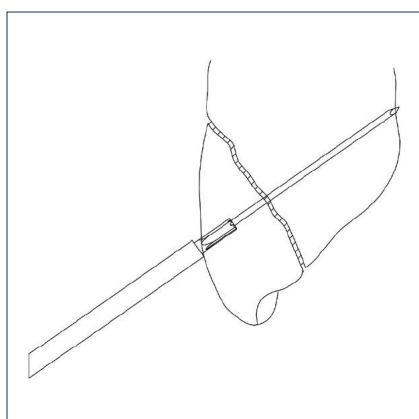
Use the cannulated proximal reamer to drill the cortex up to the stop comes into contact with the bone.

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## Surgical technique

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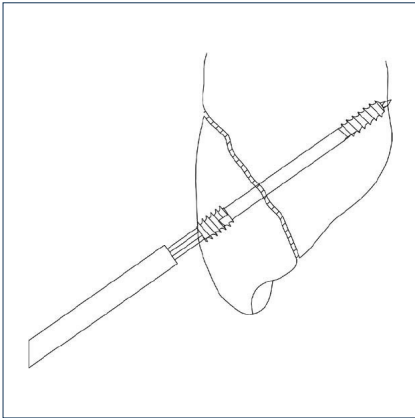Insert the screw by using the proper cannulated screwdriver.

The head of the screw must be 1 mm below the bone surface to avoid any intra-articular protrusion.

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**LSM-Med Srl**
**OVERMED Srl**